
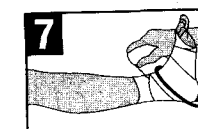
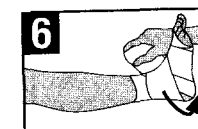
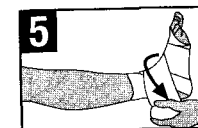
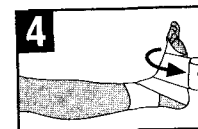
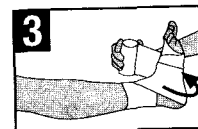
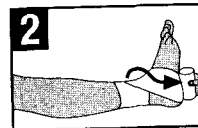
## Method 1 (drawings 1-10)

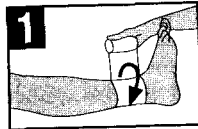**D**

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# Compridur®

**GB**

### Short stretch bandages for support, relief and compression.

**Dressing instruction:**  
We recommend that bandaging be performed by a trained person.

**Indications:**  
Compression bandages for phlebological and lymphological indications to support muscle pump action and improve the haemodynamics of venous return: varicosis, chronic venous insufficiency including venous varicose ulcer and venous oedema, thrombophlebitis, phlebothrombosis, after vein operations (stripping), after sclerosing obliteration, primary and secondary lymphatic oedemas.

**Contraindications:**  
Advanced peripheral arterial occlusive disease, decompensated cardiac insufficiency, septic phlebitis, phlegmasia coerulea dolens.

**Longitudinal elasticity:**  
Compridur®: approx. 50%  
Comprihaft®: approx. 75%  
Comprilan®: approx. 90%

**Special notes:**  
Applies only to Comprihaft®: initially unroll approximately 20-30 cm of the bandage before applying. This is the only way to guarantee a continuous drop in pressure from distal to proximal.

**Recommendation:**  
We recommend Artiflex® or Soffban® as padding material, with Leukoplast® for final securing.

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