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D

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**Lubricant eye drops.**

### Composition:

Each 1 ml of solution contains: sodium hyaluronate 1.0 mg, sodium chloride, disodium hydrogenphosphate, sodium dihydrogenphosphate, purified water, disodium edetate and polyhexanide as preservatives.

### Intended use:

For lubrication of the eyes in case of sensations of dryness, burning and ocular fatigue and other minor complaints of no pathological significance, induced, for example, by dust, smoke, dry heat, air conditioning, extended computer screen use or contact lens wear.

### Contra-Indications:

Individual hypersensitivity to any constituent of the product.

### Interactions:

Do not use VISMED® *light* at the same time as any other drug or product applied to the eye since it may modify their effects.

### Directions for use:

Twist the screw cap on the multidose container firmly clockwise, before using VISMED® *light* for the first time: a spike in the screw cap will pierce the tip of the container. Twist off the screw cap and position the tip of the container above the eye to be treated: upon slight pressure, VISMED® *light* will easily flow out from the container. If not otherwise recommended, place one or two drops of VISMED® *light* into the conjunctival sac of the eye as often as needed. After blinking, the solution will disperse and form a transparent and long lasting coating on the surface of the eye. VISMED® *light* may also be used while wearing contact lenses (rigid or soft). Solution from the open container may be used up to three months.

### Precautions for use:

Do not touch the tip of the opened container and do not touch the surface of the eye with the tip of the container. Do not use VISMED® *light* if the container is damaged. Store below 25 °C! Do not use VISMED® *light* after the expiry date indicated on the package. If discomfort persists while using VISMED® *light* or reappears after discontinuation of treatment, consult a physician. Keep out of reach of children!

### Characteristics and mode of action:

VISMED® *light* contains sodium hyaluronate, a natural polymer, which is also present in the structures of the human eye. The particular physical characteristics of sodium hyaluronate confer to VISMED® *light* its "viscoelastic" and water retaining properties. VISMED® *light* provides a stable coating on the surface of the eye which is only slowly eliminated by blinking. Therefore VISMED® *light* combines long lasting relief with maximum comfort. VISMED® *light* is well tolerated due to its unique composition. The particular composition of preservatives in VISMED® *light* guarantees the sterility of the solution in the open container for a period of three months.

**Presentation:** Multidose container of 15 ml

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