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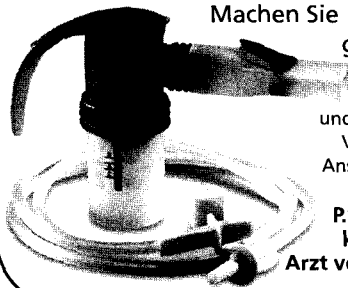
CE 0123

65NOT5300/B

EN PARI NaCl inhalation solution (0.9%) to moisten the respiratory tract

For your information

Content: 2.5ml isotonic sodium chloride solution (0.9% NaCl) - sterile

Area of use: The PARI NaCl inhalation solution is a natural, isotonic sodium chloride solution (0.9%) in a concentration adapted for compatibility with human body fluids.

The PARI NaCl inhalation solution can be used:

- as a secondary treatment for cold-like symptoms such as coughing or nasal congestion
- for naturally cleaning the respiratory tract, which has beneficial effects in the case of inflammations, very dusty or polluted atmospheres, e.g., tobacco smoke or dust
- for moistening the respiratory tract if the air in air conditioned or heated rooms is overly dry
- as a carrier solution for diluting medications that may be diluted with isotonic sodium chloride solution for inhalation according to the manufacturer's instructions (for dilution information, please read the package insert for the medication concerned)

The PARI NaCl inhalation solution is suitable for use by everyone, regardless of age. Used according to the instructions it can even be administered to infants, people of advanced years and to pregnant and nursing women.

Type of application: Put the contents of one vial into the medication cup in the nebuliser and inhale up to 4 times a day as required in accordance with the instructions of the manufacturer of your electric nozzle nebuliser (e.g., PARI inhalation devices) or ultra-sound nebuliser.

⚠ Please read the information in the instructions for use of your nebuliser and inhalation device as well!

Application temperature: Room temperature

Important information: Isotonic saline solution may cause constriction of the bronchia when inhaled by very sensitive patients such as those suffering from bronchial asthma or who have a hyperreactive bronchial system; this constriction is reversible. If this does occur, stop treatment and seek advice from your doctor or pharmacist.

This product includes small parts, which could pose a choking hazard. Always keep this product out of the reach of children aged three or younger!

Since the solution contains no preservatives, opened ampoules should be disposed of immediately for reasons of hygienic safety.

Contents and pack sizes:

2.5ml vials of PARI NaCl inhalation solution can be ordered in packs of 20 and 60.

⚠ Do not use the PARI NaCl inhalation solution after the expiration date!

Disposal: Please recycle empty ampoules.

Explanation of symbols:

	Warning	STERILE A	Symbol for sterility
	Production batch		Symbol for single use
	Expiration date		Please follow the instructions for use

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