Lomatueill® H

Salbentüll, hydrophob
Tulle gras vaseliné, hydrophobic
Paraffin Gauze Dressing, hydrophobic
Tul vaseline, hidrofob
Compresa de gaze vaselina, hidrofóbia
Tulle idropelante da unguento
Salbentüll, hydrofob
Salveteil, hydrofob
Salveteil, hydrofob
Salvkompres, vattenavvisande
Tył z mastą, hydrofobny
Tył z mastą, hydrofobny
Kenkastull, vitsatätilt
Мазевая повязка, гидрофобная
Tył z maską, hydrofobowy
凡士林紗布敷料, 防水性

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Contraindications :

Contraindications have not been observed.

Indications :

Lomatueill® H is used for the treatment of skin donor sites, leg ulcers, diabetic foot ulcers, pressure ulcers, etc.

Contraindications :

Contraindications have not been observed.

Side effects :

Side effects have not been observed.

Interactions with other preparations:

Interactions have not been observed.

Dosage and instructions for use:

After the application of Lomatueill® H, allow the absorption of the Vaseline (Paraffin). Lomatueill® H should then be applied overlapping the edges by approximately 1 cm. With heavily exuding wounds, a high absorbency dressing (e.g. Suprasorb® F) or a Vlieswickel® compress should be applied to absorb the exudate. This will protect the wound against germs and bacteria from the outside. It is also recommended to use a PU film (e.g. Suprasorb® F) to protect the wound against germs and bacteria from the outside. After the absorption of the Vaseline (Paraffin), Lomatueill® H should be applied overlapping the edges by approximately 1 cm. With heavily exuding wounds, a high absorbency dressing (e.g. Suprasorb® F) or a Vlieswickel® compress should be applied to absorb the exudate. This will protect the wound against germs and bacteria from the outside. It is also recommended to use a PU film (e.g. Suprasorb® F) to protect the wound against germs and bacteria from the outside.

Dosage charge:

After the absorption of the Vaseline (Paraffin), Lomatueill® H should then be applied overlapping the edges by approximately 1 cm. With heavily exuding wounds, a high absorbency dressing (e.g. Suprasorb® F) or a Vlieswickel® compress should be applied to absorb the exudate. This will protect the wound against germs and bacteria from the outside. It is also recommended to use a PU film (e.g. Suprasorb® F) to protect the wound against germs and bacteria from the outside.

Important information:

Lomatueill® H should not be stored above 25 °C. Lomatueill® H Paraffin Gauze Dressing is sterile, unless the package is opened or damaged. Do not resterilize.

Presentation:

Reorder Code	Sizes	Contents per box
23314	5 x 5 cm	10 pieces
23315	10 x 10 cm	10 pieces
23316	10 x 20 cm	10 pieces
23317	10 x 30 cm	10 pieces
23318	10 x 10 cm	50 pieces

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Compozycja:

Tkanina, wide-mesh, net-like tulle produced from 100 % cotton, impregnated with a hydrophobic (water-resistant) ointment basis (vaseline according to Ph. Eur.).

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