Lomatuell® H

Salbentüll, hydrofob
Tulle gras vaseline, hydrophobe
Paraffin Gauze Dressing, hydrophobic
Tül vaseline, hidrofób
Compressa de gaze vaselinată, hidrofóbă
Tulle idrorepellente da unguento
Salbentüül, hydrofob
Salvetül, hydrofob
Salkvõõrs, vattenavvisande
Tyl si masti, hydrofobni
Kenőcsőtűl, víztaszító
Мазевая повязка, гидрофобная
Tül z masťou, hydrofobový
凡士林纱布贴料, 防水性

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Instructions for use
Composition: Knitted, wide-mesh, net-like tulle produced from 100 % cotton, impregnated with a hydrophobic (water-resistant) ointment base (white petrolatum [Ph. Eur.]).
Indications: Lomatuell® H is used for the treatment of skin donor sites, leg ulcers, decubitus ulcers, burns, cuts, abrasions.
Contraindications: Contraindications have not been observed.
Side effects: Side effects have not been observed.
Interactions with other preparations: Interactions have not been observed.
Dosage and instructions for use: Prior to application, the wound should be cleaned as usual. Lomatuell® H should then be applied overlapping the wound edges by approximately 1 cm, with healthy surrounding skin. A high absorbency dressing should be applied on top (e.g. Vivazell®), with moderately to slightly fibrous tissue, or a non-woven dressing (e.g. Sani® or Vivazell®) will be sufficient. To assist a moist warm wound environment and to protect the wound against germs and bacteria from the outside, the dressing can be secured with a semi-permeable film (e.g. Suprasorb® P). On infected wounds, only a permeable dressing, repellent materials should be used (e.g. Curefit® H).
Dressing change: After application and absorption of the Vaseline (Paraffin), Lomatuell® H should be removed. As a rule, this effect lasts approximately 1-3 days.
Important information! Lomatuell® H should not be stored above 25 °C. Lomatuell® H Paraffin Gauze Dressing are sterile, unless the package is opened or damaged. Do not restreinte.

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Pacifier Code	Sizes	Contents per box
23314	5 x 5 cm	10 pieces
23315	10 x 10 cm	10 pieces
23316	10 x 20 cm	10 pieces
23317	10 x 30 cm	10 pieces
23318	10 x 10 cm	50 pieces

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Lohmann & Rauscher International GmbH & Co. KG, D-65779 Hepsigdorf