Lomatuell® H

- Sabimentu, hidrófobu
- Tulle gras vaseline, hidrófobu
- Paraffin Gauze Dressing, hidrófobu
- Tul vaseline, hidrófobu
- Compressa de gaze vaselinada, hidrófobu
- Tulle idrófobu de ungento
- Sabimentu, hidrófobu
- Sabimentu, hidrófobu
- Sabimentu, hidrófobu
- Salvcompres, vattenavvisande
- Tyl i masti, hidrófobny
- Tyl i masti, hidrófobny
- Kendocstül, vízszázó
- Мазевна повязка, гидрофобная
- Tül z maszczą, hydrofobowy
- 凡士林紗布敷料, 防水性

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Instructions for use

Composition: cotton, white mesh, non-tulle tulle produced from 100 % cotton, impregnated with a hydrophobic (water-repellant) ointment basis (white petrolatum (Ph. Eur.).
Indications: Lomatuell® H is used for the treatment of skin donor sites, leg ulcers, decubitus ulcers, burns, cuts, abrasions.
Contraindications: Contraindications have not been observed.
Side effects: Side effects have not been observed.
Interactions with other preparations: Interactions have not been observed.
Dosage and instructions for use: Prior to application, the wound should be cleaned as usual. Lomatuell® H should then be applied overlapping the wound edges by approximately 1 cm. With heavy exudate (e.g. Vivazole®), with moderate to light exudate (e.g. Gauze® or Vivazole®) or with dry wounds, a gauze or nonwoven dressing (e.g. Gauze® or Vivazole®) will be sufficient. To assist a moist warm wound environment and to protect the wound against germs and bacteria from outside, the dressing can be secured with a semi-permeable film (e.g. Suprasec® F). On infected wounds, only permeable dressing materials should be used (e.g. Curafix® H).**Dressing change:** After liquefaction and absorption of the Vaseline (Paraffin), Lomatuell® H should be removed. As a rule, this effect takes approximately 1-3 days.
Important information! Lomatuell® H should not be stored above 25 °C. Lomatuell® H Paraffin Gauze Dressings are sterile, unless the package is opened or damaged. Do not sterilize.

Important information!

Lomatuell® H should not be stored above 25 °C. Lomatuell® H Paraffin Gauze Dressings are sterile, unless the package is opened or damaged. Do not sterilize.

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| Receptor Code | Size | Contents per box |
|---------------|------------|------------------|
| 23314 | 5 x 5 cm | 10 pieces |
| 23315 | 10 x 10 cm | 10 pieces |
| 23316 | 10 x 20 cm | 10 pieces |
| 23317 | 10 x 30 cm | 10 pieces |
| 23318 | 10 x 10 cm | 50 pieces |

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Lohmann & Rauscher International GmbH & Co. KG · D-56579 Rengsdorff