VIS MED®

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Composition: Each 1ml of solution contains: sodium hyaluronate 1.8 mg, sodium chloride, potassium chloride, disodium-hydrogenphosphate, sodium citrate, magnesium chloride, calcium chloride and water for injections. The solution is hypotonic.

Intended use: For lubrication of the eyes in case of sensation of dryness, burning and ocular fatigue and other minor complaints of no pathological significance, induced, for example, by dust, smoke, dry heat, air conditioning, wind, cold, extended computer screen use or contact lens wear (rigid or soft).

Contra-Indications: Individual hypersensitivity to any constituent of the product.

Interactions: Do not use VISMED® multi at the same time with any drug or other product applied to the eye since it may modify their effects.

Directions for use: The VISMED® multi container is equipped with an innovative dosage system, which facilitates the instillation of the drop into the eye. Carefully read through the instructions for use below. If not otherwise recommended, place 1–2 drops of VISMED® multi into the eye as often as needed.

Precaution for use: Do not touch the aperture of the dosage system and do not touch the surface of the eye with the aperture. Do not use VISMED® multi if the container is damaged. VISMED® multi can be used up to three months after first use. Store between 2 °C and 25 °C! Do not use VISMED® multi after the expiry date. If discomfort persists while using VISMED® multi or reappears after discontinuation of administration, consult a physician. Keep out of reach of children.

Characteristics and mode of action: VISMED® multi contains sodium hyaluronate, a natural polymer, which is also present in the structures of the human eye. The particular physical characteristics of sodium hyaluronate confer to VISMED® multi its "viscoelastic" and water retaining properties. VISMED® multi provides a stable coating on the surface of the eye which is only slowly eliminated by blinking. Therefore VISMED® multi combines long lasting relief with maximum comfort. VISMED® multi is well tolerated due to its unique composition. It is free from preservatives.

Presentation:

10 ml preservative-free multidose container with an innovative dosage system.
Last revision date: August 2006

Legend to the pictures 1-5:

- 1 Remove the protective cap from the container.
- 2 Place the container upside down between thumb and index finger ensuring that the thumb is on the head of the container. Prior to the first use, the mechanism has to be operated several times until ejection of the first drop of solution. In order to do this, apply pressure to the container with your index finger.
- 3 Pull down the lower lid of the eye with the fingers of your free hand.
- 4 Position the container by resting your thumb onto the fingers holding the lid, ensuring that the aperture is directly over the opened eye.
- 5 Tilt the container to an angle of approximately 45° to your face, as the drop will leave the container at this angle. Apply a uniform downward pressure to the container with the index finger to instill a drop of solution into the eye. You may need several attempts at first but you will learn the technique easily, as it is always the same action and positioning on the face.

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