Rosidal® K

100% CO

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Durelast®

66% CO, 34% PA

Short stretch bandages for compression, support and relief bandaging
Instructions for Use
Indications:

- **Compression bandaging** for phlebological and lymphological indications, e.g. varices, insufficiency of the perforator veins, thrombophlebitis, phlebothrombosis, post-thrombotic syndrome, chronic venous insufficiency stages I to III, in cases of oedemas (phleboedema, lymphoedema, lipoeidema, posttraumatic, postoperative, during pregnancy), after varicose vein procedures.
- **Support and relief bandaging** for the prophylaxis and treatment of contusions, distorsions, tendon injuries and luxations.

Contraindications:
 Advanced peripheral occlusive arterial disease, decompensated heart insufficiency, septic phlebitis, phlegmasia coerulea dolens.

Important notes:
Rosidal® haft must be unrolled about 20 cm prior to application. **Elko®-Rosidal® K** - For compression bandaging in double bandage techniques (fig. A), the second Rosidal® bandage must be applied over the first one in the opposite direction using the same technique. Start either at the ankle joint or at the base of the toes (see fig. A10 and A11).

Durelast® Combi may be applied according to Sigg's compression bandaging technique (fig. B). To start bandaging, use the 8 cm wide Durelast®. From the calf onwards, the 10 cm bandage is applied.

- General notes:**
1. Ideally apply the bandage when the swelling is down, preferably after the patient has just got up in the morning or after having rested the leg in a raised position for some time.
 2. Select a bandage which is equal in width to the diameter of the limb to be treated. Bandages, 8, 10 or 12 cm wide, are most commonly used.
 3. Unless otherwise indicated start bandaging on the foot which should form an angle of 90° with the leg.
 4. Apply the bandage at consistent pressure to ensure graduated compression from the foot to the knee.
 5. Apply the bandage holding the roll close to the limb and pointing outward.
 6. Ask the clinician to indicate the particular bandaging technique he or she wishes to be applied.
 7. Inexpert bandaging may lead to cutaneous gangrene and nervous pressure injuries.

Special washing instructions for Rosidal® haft:
 Before washing, position in zig-zag winds measuring approx. 20 cm and squeeze together.

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Lohmann & Rauscher International GmbH & Co. KG, D-56579 Rengsdorf