
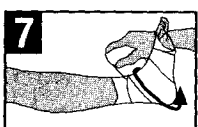
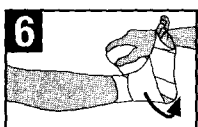
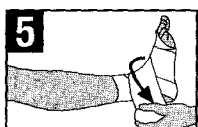
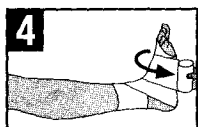
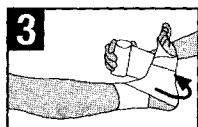
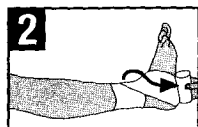
Method 1 (drawings 1-10)

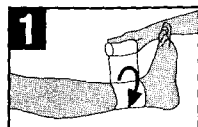Compridur®

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Comprilan®

GB
Short stretch bandages for support, relief and compression.

Dressing instruction:
We recommend that bandaging be performed by a trained person.

Indications:
Compression bandages for phlebological and lymphological indications to support muscle pump action and improve the haemodynamics of venous return: varicosis, chronic venous insufficiency including venous varicose ulcer and venous oedema, thrombophlebitis, phlebothrombosis, after vein operations (stripping), after sclerosing obliteration, primary and secondary lymphatic oedemas.

Contraindications:
Advanced peripheral arterial occlusive disease, decompensated cardiac insufficiency, septic phlebitis, phlegmasia coerulea dolens.

Longitudinal elasticity:
Compridur®: approx. 50%
Comprihaft®: approx. 75%
Comprilan®: approx. 90%

Special notes:
Applies only to **Comprihaft®**: initially unroll approximately 20-30 cm of the bandage before applying. This is the only way to guarantee a continuous drop in pressure from distal to proximal.

Recommendation:
We recommend **Artiflex®** or **Soffban®** as padding material, with **Leukoplast®** for final securing.

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